

GeneXpert Infinity Quiz

Name of Trainee: _____

Date: _____

Please select as many answers that apply

1. When would you push the emergency stop?



- a. A cartridge is stuck or fallen
- b. To remove a retained cartridge
- c. To shut down the instrument
- d. To clean the conveyor
- e. A spill has occurred on the conveyor


2. What do you use to clean the conveyor belt?

- a. 10% bleach
- b. 70% Alcohol
- c. Paper towel
- d. Deionized water

Please put into the correct order

3. How do you shut down the Infinity system?

Step 1 ___ Step 2 ___ Step 3 ___ Step 4 ___

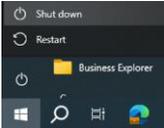- a. Select the Icon on the Windows Desk top screen that says System Shutdown 
- b. Turn off the Infinity power switch
- c. Make sure nothing is running then select the Exit Icon 
- d. Wait 2 minutes
- e. Select Shut Down from the Windows Start button 

Please provide a short answer:

- 4. Name one way to view the time remaining until a test is complete.
- 5. When accessing the waste bin area, why is it important to state whether or not the waste bin has been emptied?
- 6. Briefly describe how to access or remove a cartridge from the accumulator shelf.

GeneXpert Infinity Quiz

7. Please use the numbers on the right to label appropriate part of photo on left.



1-Kiosk

2-Conveyor Belt

3-Barcode Scanners

4-Shuttle

5-Waste Container

6-Emergency Stop Button

7-Modules

8-Accumulators

9-Gantry

10-Power Switch

11-Lobby

GeneXpert Infinity Quiz

8. Place the appropriate Icon number next to the corresponding description.

1	
2	
3	
4	
5	
6	
7	
8	

	Manual mode operation
	Indicates outstanding order needs attention
	Power down the Infinity
	Login
	Error is present
	Exit software
	View Another Test
	Cartridge I/O

GeneXpert Infinity Quiz – Answer Key

Name of Trainee: _____

Date: _____

Please select as many answers that apply

1. When would you push the emergency stop?



- a. A cartridge is stuck or fallen
- b. To remove a retained cartridge
- c. To shut down the instrument
- d. To clean the conveyor
- e. A spill has occurred on the conveyor

2. What do you use to clean the conveyor belt?

- a. 10% bleach
- b. 70% Alcohol
- c. Paper towel
- d. Deionized water

Please put into the correct order

3. How do you shut down the Infinity system?

Step 1 c Step 2 a Step 3 d Step 4 b

- a. Select the Icon on the Windows Desk top screen that says System Shutdown




- b. Turn off the Infinity power switch

- c. Make sure nothing is running then select the Exit Icon



- d. Wait 2 minutes

- e. Select Shut Down from the Windows Start button

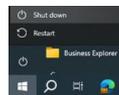

Please provide a short answer:

- 4. Name one way to view the time remaining until a test is complete.



View Another Test icon



Home Screen Icon

- 5. When accessing the waste bin area, why is it important to state whether or not the waste bin has been emptied?


When you confirm that the waste container is empty, the waste counter will reset to zero cartridges. If there are cartridges remaining when the counter is re-set, the existing cartridges will not be included in the total. The system will not allow new tests to be ordered when the cartridge count is 400.

- 6. Briefly describe how to access or remove a cartridge from the accumulator shelf.

- 1. Select Cartridge I/O.
- 2. Select the check box next to the cartridge(s) that you want to remove.
- 3. Select Move to Shuttle.
- 4. Select Open Shuttle.
- 5. The shuttle door will unlatch.
- 6. Carefully pull the shuttle door open and remove the cartridge(s).
- 7. Close the shuttle door and select OK.

GeneXpert Infinity Quiz

7. Please use the numbers on the right to label appropriate part of photo on left.

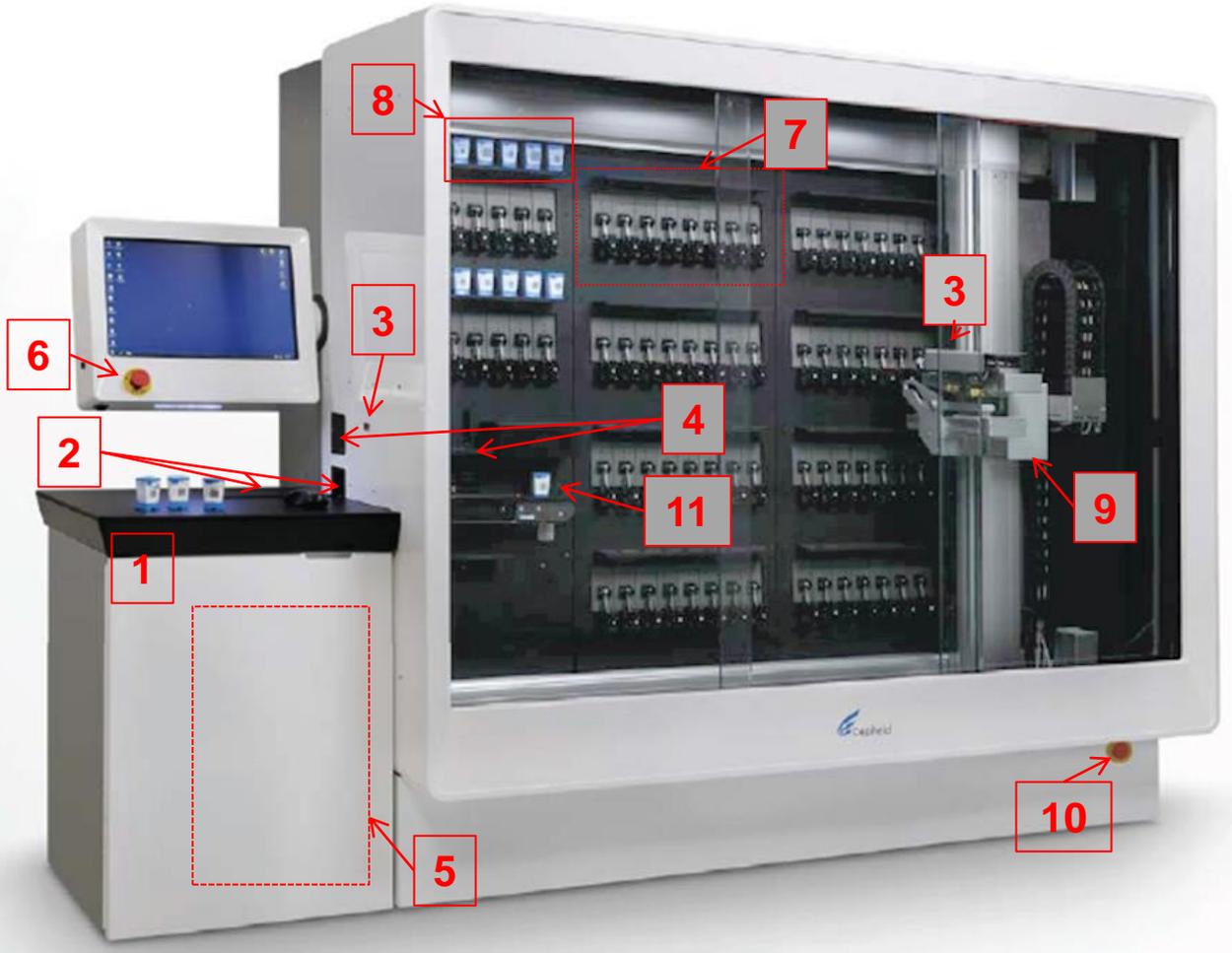

1-Kiosk

2-Conveyor Belt

3-Barcode Scanners

4-Shuttle

5-Waste Container

6-Emergency Stop Button

7-Modules

8-Accumulators

9-Gantry

10-Power Switch

11-Lobby

GeneXpert Infinity Quiz

8. Place the appropriate Icon number next to the corresponding description.

1	
2	
3	
4	
5	
6	
7	
8	

5	Manual mode operation
6	Indicates outstanding order needs attention
7	Power down the Infinity
1	Login
3	Error is present
2	Exit software
8	View Another Test
4	Cartridge I/O