English

For *in vitro* diagnostic use only.

Rx Only

Intended Use

The Cepheid® Xpert® Urine Specimen Collection Kit is designed to preserve and transport *Chlamydia trachomatis*, *Neisseria gonorrhoeae*, and *Trichomonas vaginalis* DNA in first-catch female and male urine specimens from symptomatic and asymptomatic individuals prior to analysis with the Xpert CT/NG Assay and the Xpert TV Assay.

Summary and Explanation

Refer to the package insert for the Xpert CT/NG Assay or the Xpert TV Assay (as appropriate).

Principles of the Procedure

The Xpert Urine Specimen Collection Kit allows for an extended range of time and temperature conditions for specimen storage and transport when testing for the presence of *Chlamydia trachomatis*, *Neisseria gonorrhoeae*, or *Trichomonas vaginalis* in urine specimens using the Cepheid Xpert CT/NG Assay or the Xpert TV Assay (as appropriate). Approximately 7 mL of first-catch urine is transferred from the collection cup to the Xpert Urine Transport Reagent tube using a disposable transfer pipette.

Reagents

Materials Provided

Xpert Urine Specimen Collection Kits: (quantity: 50)

Each kit contains a sealed package containing:

- 1 tube containing Xpert Urine Transport Reagent (yellow cap)
- 1 sterile disposable transfer pipette (individually wrapped)
- Instructions for transfer of urine to collection kit (quantity: 1)

Materials Required but Not Provided

- Clean, plastic, preservative-free, urine collection cups
- Disposable gloves
- Adhesive labels or sample identification information
- Xpert CT/NG Assay or the Xpert TV Assay

Urine Specimens Appropriate for Use

The table below identifies urine specimen types and assay targets that are acceptable for use with the following assays.

Xpert Assay	Assay Target	First-Catch Female Urine Specimen	First-Catch Male Urine Specimen
Xpert CT/NG Assay	<i>Chlamydia trachomatis</i> and <i>Neisseria gonorrhoeae</i>	Yes	Yes
Xpert TV Assay	<i>Trichomonas vaginalis</i>	Yes	Yes

Warnings and Precautions

- For prescription use only.
- Treat all biological specimens, including used cartridges, as if capable of transmitting infectious agents. Because it is often impossible to know which might be infectious, all biological specimens should be treated with standard precautions. Guidelines for specimen handling are available from the U.S. Center for Disease Control and Prevention and the Clinical and Laboratory Standards Institute.^{1,2}
- Avoid cross contamination during sample handling steps.
- Do not use kit if opened or damaged.
- Spilled or leaking transport reagent tubes should be discarded and not used.
- Use this collection kit only with the Xpert CT/NG Assay or the Xpert TV Assay.
- Follow your institution's safety procedures for working with chemicals and handling biological samples.
- With urine specimens tested with the Xpert CT/NG Assay, assay interference may be observed in the presence of: blood (>0.3% v/v), mucin (>0.2% w/v), bilirubin (>0.2 mg/mL), or Vagisil feminine powder (>0.2% w/v).

Kit Storage Requirements

Store collection kit at 2 °C to 30 °C.

IMPORTANT: Do not use after the expiration date which is clearly printed on the outer box label.

Limitations

Use this collection kit only with the Xpert CT/NG Assay or the Xpert TV Assay.

Specimen Collection and Handling

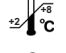
1. The patient should not have urinated for at least 1 hour prior to specimen collection.
Female patients should not cleanse the labial area prior to collecting the specimen. Male patients should not cleanse the tip of the penis prior to collecting specimen.
2. Direct patient to provide first-catch urine (approximately 20 to 50 mL of the initial urine stream) into a urine collection cup free of any preservatives. Collection of larger volumes of urine may result in specimen dilution that may reduce test sensitivity.
3. Ensure that the urine is well mixed in the urine cup before transferring a sample to the Xpert Urine Transport Reagent tube.
4. Open the packaging of a disposable transfer pipette provided in the kit.
5. Remove the cap from the Xpert Urine Transport Reagent tube and from the urine collection cup.
6. Insert the transfer pipette into the urine cup so that the tip is near the bottom of the cup. Transfer approximately 7 mL of urine into the Xpert Urine Transport Reagent tube using the disposable transfer pipette. The correct volume of urine has been added when the level reaches the black dashed line on the label of the Xpert Urine Transport Reagent tube.
7. Replace the cap on the Xpert Urine Transport Reagent tube and tighten securely.
8. Invert the reagent tube 3-4 times to ensure that the specimen and reagent are well mixed.

9. Recap the urine cup securely.
10. Label the transport tube with sample identification information, including the date of collection, as required. Take care not to obscure the fill line on the Xpert Urine Transport Reagent tube.
11. Transport and store the female or male urine samples in the Xpert Urine Transport Reagent tube according to the conditions indicated in Table 2 for the Xpert CT/NG Assay or for the Xpert TV Assay (as appropriate).

Specimen Transport and Storage

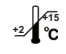
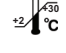
Maintain proper storage conditions during specimen transport to ensure the integrity of the specimen. Specimen stability under shipping conditions other than those recommended below in Table 1 and Table 2 has not been evaluated.

Table 1. Unprocessed Urine Specimen*

Specimen	Xpert Assay	Transport and Storage Temperature (°C)	Storage Time
 Female Urine	Xpert CT/NG Assay	4 °C	8 days
		15-30 °C	24 hours
 Male Urine	Xpert CT/NG Assay	4 °C	8 days
		15-30 °C	3 days
 Female and Male Urine	Xpert TV Assay	2-8 °C	4 days
		15-30 °C	4 hours

* Unprocessed urine can be stored and transported under these conditions prior to addition to the Cepheid Urine Transport Reagent. The Xpert CT/NG Assay and the Xpert TV Assay are intended for use only with urine specimens transferred to the Urine Transport Reagent.

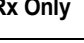
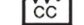
Table 2. Urine Specimens in Xpert Urine Transport Reagent

Specimen	Xpert Assay	Transport and Storage Temperature (°C)	Storage Time (days)
 Female Urine in Xpert Urine Transport Reagent	Xpert CT/NG Assay	2-15 °C	45 days
		2-30 °C	3 days
 Male Urine in Xpert Urine Transport Reagent	Xpert CT/NG Assay	2-30 °C	45 days
 Female and Male Urine in Xpert Urine Transport Reagent	Xpert TV Assay	2-8 °C	28 days
		15-30 °C	14 days

References

1. Centers for Disease Control and Prevention. Biosafety in microbiological and biomedical laboratories (refer to latest edition). www.cdc.gov/biosafety/publications. www.cdc.gov/biosafety/publications
2. Clinical and Laboratory Standards Institute. Protection of laboratory workers from occupationally acquired infections; Approved Guideline. Document M29 (refer to latest edition).

Table of Symbols

Symbol	Meaning
	Catalog number
	CE marking – European Conformity
	<i>In vitro</i> diagnostic medical device
	Contains sufficient for <n> tests
	Sterilized using irradiation
	Authorized representative in the European Community
	Do not reuse
	Manufacturer
	Temperature limitation
	Batch code
	Caution
	For prescription use only
	Consult instructions for use
	Expiration date
	Biological risks
	Country of Manufacture

Technical Support Contact

Region	Telephone	Email
US	+1 888.838.3222	techsupport@cepheid.com
Australia and New Zealand	+1800 130 821 +0800 001 028	techsupportanz@cepheid.com
Belgium, Netherlands and Luxembourg	+33 563 825 319	support@cepheideurope.com
Brazil and Latin America	+55 11 3524 8373	latamsupport@cepheid.com
China	+86 400 821 0728	techsupportchina@cepheid.com
France	+33 563 825 319	support@cepheideurope.com
Germany	+49 69 710 480 480	support@cepheideurope.com
India, Bangladesh, Bhutan, Nepal and Sri Lanka	+91 11 48353010	techsupportindia@cepheid.com
Italy	+39 800 902 567	support@cepheideurope.com
Portugal	+351 800 913 174	support@cepheideurope.com
Spain	+34 919 90 67 62	support@cepheideurope.com
South Africa	+27 861 22 76 35	support@cepheideurope.com
United Kingdom	+44 3303 332 533	support@cepheideurope.com
Other European, Middle East and African countries	+33 563 825 319 +971 4 253 3218	support@cepheideurope.com
Other countries not listed above	+1 408.400.8495	techsupport@cepheid.com

Cepheid
904 Caribbean Drive
Sunnyvale, CA 94089
USA
Phone: +1.408.541.4191
Fax: +1.408.541.4192

Cepheid Europe SAS
Vira Solelh
81470 Maurens-Scopont
France
Phone: +33 563 825 300
Fax: +33 563 825 301

For Information Only - Not a Controlled Copy

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- Centers for Disease Control and Prevention. Biosafety in microbiological and biomedical laboratories (refer to latest edition). www.cdc.gov/biosafety/publications. www.cdc.gov/biosafety/publications
- Clinical and Laboratory Standards Institute. Protection of laboratory workers from occupationally acquired infections; Approved Guideline. Document M29 (refer to latest edition).

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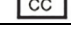
- Centers for Disease Control and Prevention. Biosafety in microbiological and biomedical laboratories (参考最新版). www.cdc.gov/biosafety/publications.
- Clinical and Laboratory Standards Institute. Protection of laboratory workers from occupationally acquired infections; Approved Guideline. Document M29 (参考最新版)。

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This Page is Intentionally Left Blank.

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This Page is Intentionally Left Blank.

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This Page is Intentionally Left Blank.

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